V8491

EN

Heat Pack – Instructions for us

First time use: when you receive your Heat Pack the liquid should be in a liquid state. If however the liquid has turned to a solid state during transport, please place the pack in boiling water for 6-8 minutes until it becomes clear liquid. Allow to cool.

To apply heat, locate the metal disc inside the liquid and squeeze it several times. The liquid will begin to turn solid and emit heat immediately.

Do not apply the Heat Pack directly to the skin, but place it on a towel for the first 10 minutes in order to avoid a burn of the skin.

The Heat Pack should emit shooting heat for 30-45 minutes and will gradually cool.

If further treatment is required boil the Heat Pack for 6-8 minutes until it becomes clear liquid again and allow cooling before reactivating the Heat Pack by squeezing the metal disc.

Do NOT squeeze the metal disc when the Heat Pack is in a solid state.

Warnings

- This item is for external use only
- Avoid sharp objects and heavy weight pressure
- Keep away from children
- If contents are swallowed, drink 1 liter of water and seek medical advice
- If contents are accidentally splashed on eyes or skin, rins immediately and thoroughly with clean water.

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